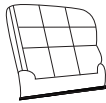



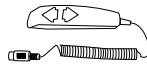
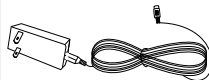


ASSEMBLY INSTRUCTION FOR POWER MOTION LIFT CHAIR U-4610-8490-108

Caution: Please read instructions thoroughly before unpacking assembly parts. Sharp, exposed staple tips can cause injury, therefore, for your protection, please remove any exposed staples used in packing.

1. We recommend that you should assemble this product with the assistance of another person; this will make assembly easier, and will help to eliminate damage to the product or injury to persons during assembly.
2. Be sure to check all packing materials carefully for small parts that may have come loose inside the carton during shipment.
3. Please do not over tighten screws or bolts.
4. Please put all parts on a non-abrasive floor before assembly, and follow the assembly steps to assemble your newly purchased product correctly and efficiently.

PARTS LIST				
NO.	PART DESC		CODE	QTY
A	BACK REST			1
B	SEAT			1
C	WING(E)			1
D	WING(F)			1

HARDWARE LIST				
NO.	PART DESC		CODE	QTY
A	Control Wand		PU-4610-RMT	1
B	Power Adapter		PU-4610-ADT	1

Do not plug your recliner power cord into a power outlet until your recliner is completely assembled and in the location of its intended use.

Notes:


- 1) To help prevent shock or damage to your product, only plug the power cord into a grounded outlet.
- 2) Read the operating instructions carefully before operating your power sofa.
- 3) Mechanism weight limited 350 LBS.

ASSEMBLY INSTRUCTION FOR POWER MOTION LIFT CHAIR U-4610-8490-108

Please following the drawings below to assemble the item


STEP:1

Insert back rest. Make sure the brackets are secured in correct position.

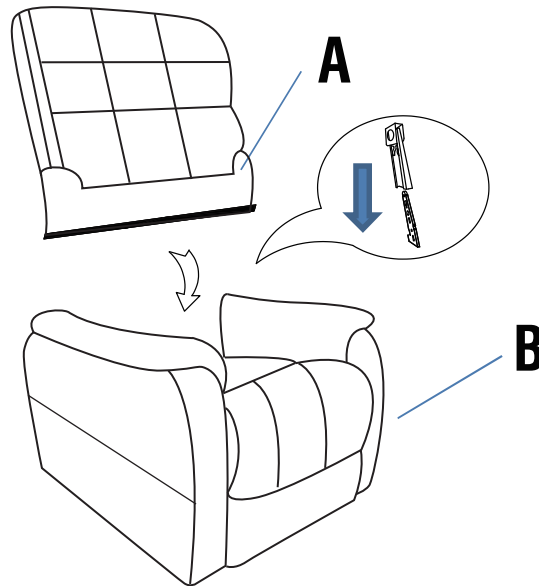

STEP:2


Attaching the KD Wings.

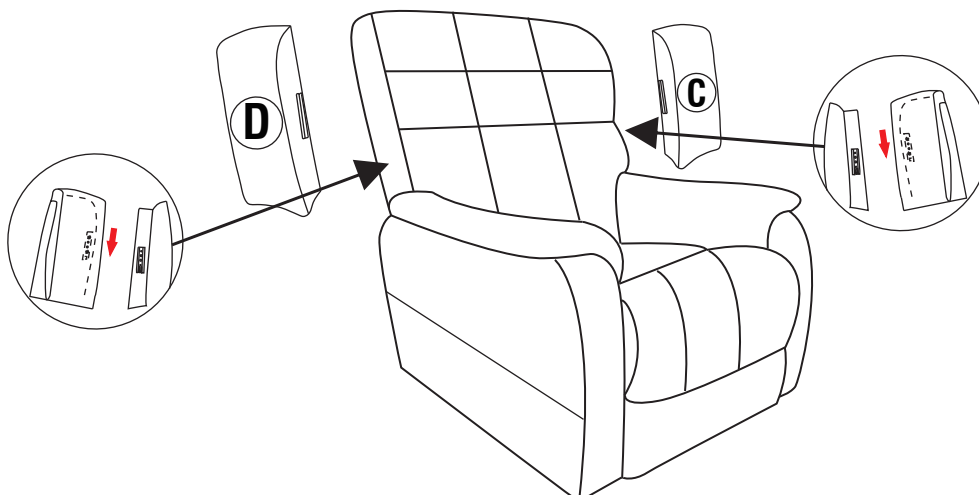

ASSEMBLY INSTRUCTION FOR POWER MOTION LIFT CHAIR U-4610-8490-108


STEP:3

Completed.

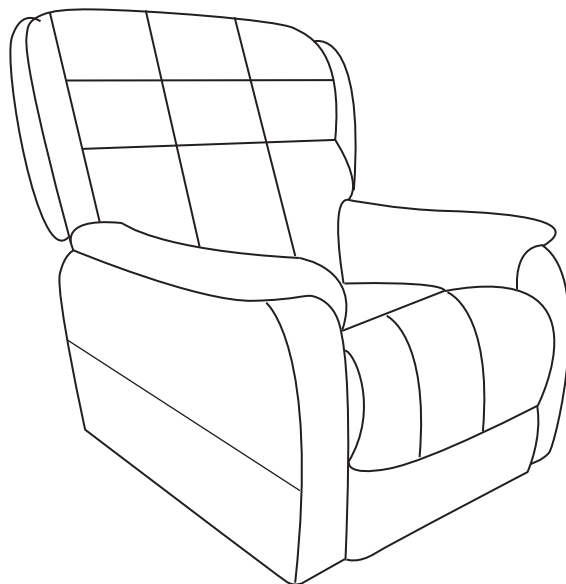



STEP:4

The packaging bag found inside the carton includes the Control Wand, Power Adapter. Remove from the packaging bag to begin assembly.

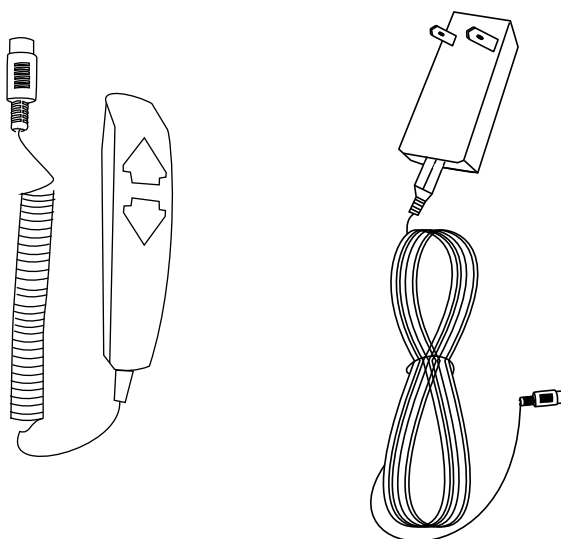



ASSEMBLY INSTRUCTION FOR POWER MOTION LIFT CHAIR U-4610-8490-108

Attach the control wand pin plug to the matching pin plug at the rear of the seat. Make sure to lock the 2 pin plugs together using the snap lock. Follow the routing of the control wand wiring as seen beside.

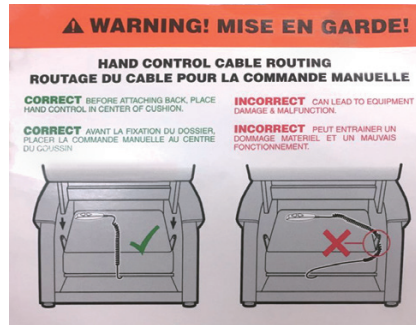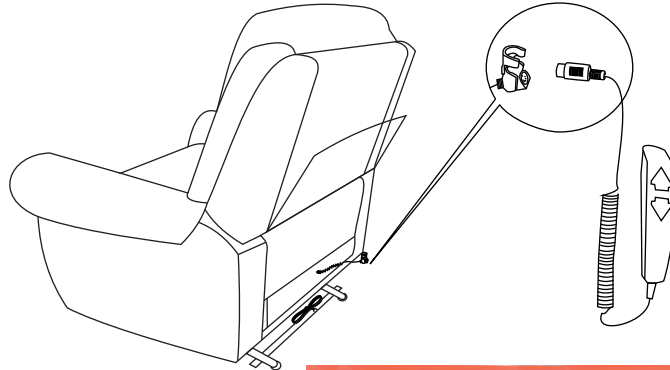

STEP:5

Attach the lead wire from the power motor found attached at the rear of the lift chair to the power adapter.

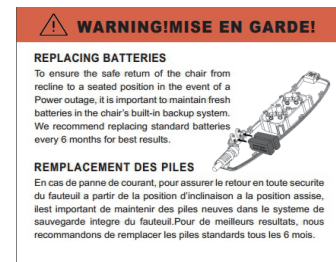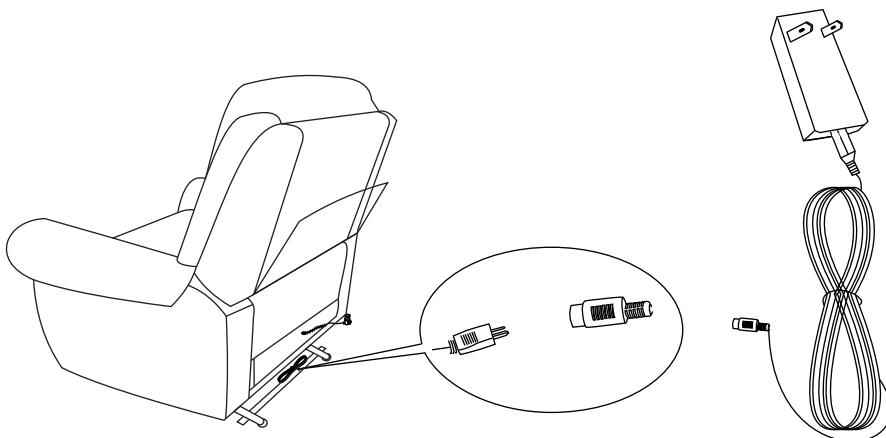

COMPLETED