HOCKER' FURNISHINGS ASSEMBLY INSTRUCTION ITEM: RC611-PHLL4-010 Flynn Power Recliner w/ PH, Lumbar, and Lift

Thank you for purchasing this quality product. Be sure to unpack carefully looking for small parts which may have come loose during shipment. Read all instruction and study the drawing carefully before starting the assembly process.

CAUTION: Make sure to read the instructions below before assembling your four motor lift chair. We suggest two people be used in the assembly of your lift chair.



Do Not plug the power cord into a power outlet until your recliner is fully assembled and in place. Notes:

1) To help prevent shock and or damage to your product only plug the power cord into a grounded outlet.

2) Read the operating instructions carefully before operating your lift chair.

IMPORTANT NOTE

- To avoid any risk of suffocation to children or pets, dispose of all plastic bags immediately.
- To ensure proper performance, occasionally check for loose screws and tighten as needed.

FURNISHINGS OPERATING INSTRUCTION ITEM: RC611-PHLL4-010 Flynn Power Recliner w/ PH, Lumbar, and Lift

Thank you for purchasing this quality product. Be sure to unpack carefully looking for small parts which may have come loose during shipment. Read all instruction and study the drawing carefully before starting the assembly process.

CAUTION:Read the important instructions below before operating your power mechanized equipment Do not allow children to play on or operate this power mechanized equipment



NOTE: This device complies with part 15 of the FCC rules. Operation is subject to the following two conditions: (1) this device may not cause interference, and (2) this device must accept any interference, including interference that may cause undesired operation of the device. Modifications not approved by the party responsible for compliance could void user's authority to operate the equipment.

Customer Service

H

If you have any questions regarding the use of this unit, or find that your unit needs service, please contact your retailer.

Product Protected By Following Patents: Patented Articulating Power Headrest & Lumbar US Patent: 10,736,423 00887A Page 2 of 2